



VENDLET

the electric turning aid

VENDLET makes repositioning of patients an easy one-person job. The touch of a button is all it takes.



VENDLET gently repositions the patient in bed while sparing the nurse's back

VENDLET matches modern-day requirements to quality, safety and efficiency in patient care by relieving caregivers of manual lifting and strain when handling patients in bed.

Benefits include:

- Prevention of caregiver injuries
- Ease of daily care routines
- Increased patient comfort
- Positive contact between caregiver and patient
- Improved caregiver productivity
- Easy attachment to most healthcare and hospital beds

How VENDLET works

The continuous turning sheet is rolled onto two motorised bars, one on each side of the bed. The bars revolve when activated by the hand control. As the turning sheet tightens firmly on the bar, the patient is repositioned or turned.



Turning and repositioning



Using a lifting sling



Washing and changing



The caregiver may optimize the working posture by lowering the bars, drawing the patient close to the edge and adjusting the height of the bed.



The turning sheet supports the full length of the body, allowing even and gentle handling of the patient. VENDLET also reduces friction and shear which can help minimize the risk of decubitus.



The patient is handled with ease and dignity. This in turn promotes job satisfaction for the caregiver and well-being for the patient.

VENDLET models

	VENDLET V4	VENDLET HC-2
Max. capacity	200 kg (400 kg*)	200 kg
Turning function	Electric	Electric
Raising and lowering of bars	Electric	Manual
Space requirement along the sides of the bed	None	Min. 1 m
Use of safety locks	None	Manual
Attachment of side rails	Manual	Manual




The built-in timer disconnects after 1½ minutes. Press the green start button to activate the timer again.

* VENDLET V4 can be custom-built for users weighing up to 400 kg.

Technical specifications

Power supply	Transformer
Rated power	100 VA
Rated voltage	Primary 230 Volt AC/50HZ Secondary 27 Volt DC
Compliance	European Medical Device Directive 93/42/EEC

All rights reserved.
Copyright 2009 H.C. Equipment ApS

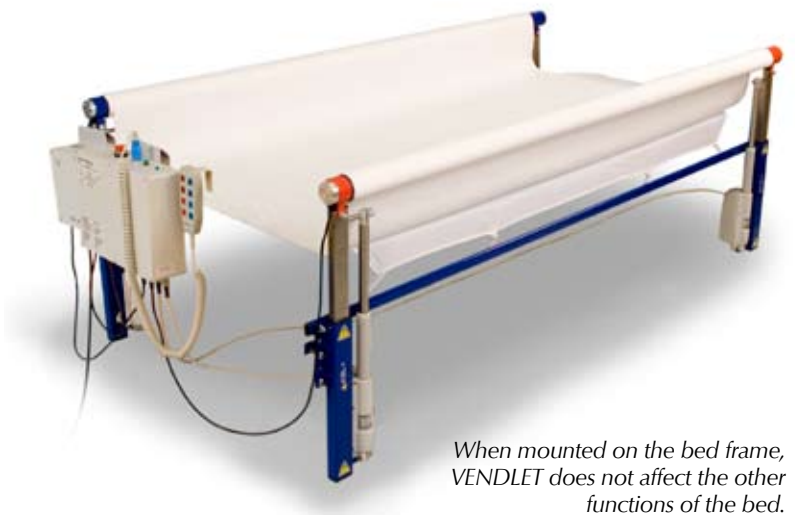

When mounted on the bed frame, VENDLET does not affect the other functions of the bed.

Distributor: