Moving Health Forward

**EN** [User Manual](#)

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# LEJRELET

## Tube 250

# EN

## Application


LEJRELET Tube 250 is a long soft cylinder filled with pressure relieving viscoelastic foam granules.

It is easy to shape and adapt Tube 250 for the individual user and care situation and thus possible to adapt Tube 250 to various situations and conditions.


The viscoelastic foam granules give great adaptability to the body and good support / pressure prevention. Further more, Tube 250 is very comfortable and calming for the bedridden person.

Examples for use of LEJRELET Tube 250:


- Support of the entire body in lateral position
- Positioning in a 30 degree angle
- Pressure relief for scapula and arms in supine position
- Support of legs and pressure relief of heels
- Comfort and calming positioning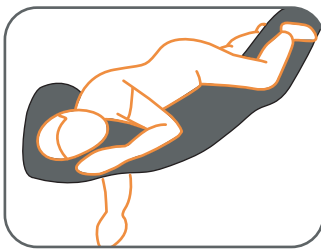


*Support of the entire body in lateral position*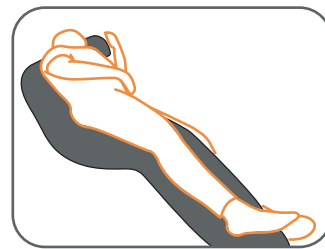

*Positioning in a 30 degree angle*

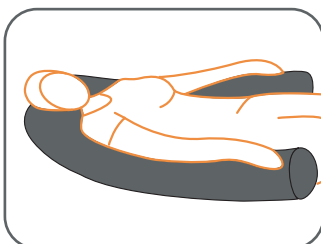*Pressure relief for scapula and arms*

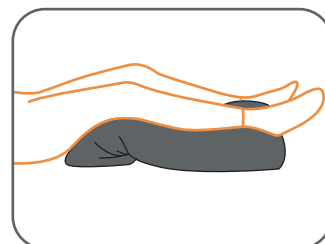*Support of legs and pressure relief of heels*

## Technical Information

### Filling:

LEJRELET Tube 250 is filled with viscoelastic foam granules preventing decubitus.

### Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

## Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

**Size:** Ø 25 x L 250

**Item no.:** 5000001

**Weight:** 4700 g

Be aware that the patient is positioned such that airways are kept clear.

## Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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**This Page is Left Intentionally Blank**

**This Page is Left Intentionally Blank**

Moving Health Forward

**PRESSURE ULCER PREVENTION**  
**SAFE MOVING & HANDLING**  
**SPECIALIST THERAPIES**  
**BATHROOM SAFETY**  
**RENTAL & SERVICE SOLUTIONS**

Direct Healthcare Group Ltd  
Withey Court, Western Industrial Estate  
Caerphilly, United Kingdom  
CF83 1BF

T: +44 (0) 800 043 0881  
F: +44 (0) 845 459 9832  
E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)

**EC** **REP**

**EU Authorised Representative, Importer, and Distributor:**

Direct Healthcare Group Sverige AB,  
Torshamnsgatan 35, SE-164 40, Kista  
Sweden  
T: +46 (0) 8 557 200

**Australian Sponsor:**

Direct Healthcare Group Pty Ltd.,  
68 Howe Street, Osborne Park,  
Western Australia 6017  
T: +61 (0) 423 852 810

Issue 3  
Date: June 2022

[DHG-HEALTHCARE.COM](http://DHG-HEALTHCARE.COM)